

THE CLEAN CUT

ULTRA-LOW

CARE & USE

Thank you for your purchase of The CleanCut Ultra-Low™. CleanCut has prepared these standard care and use instructions for the The CleanCut Ultra-Low™. Please feel free to contact the installer of this product or CleanCut, with any questions you may have. Thank you for your purchase of The CleanCut Ultra-Low™!

- *Do not use the tub or apply weight to The CleanCut Ultra-Low™ for two (2) days after installation to allow for the Clean adhesive to completely dry.*
- *Prior to first use, please inspect the bead of caulk around the perimeter of The CleanCut Ultra-Low™ to ensure that a continuous caulk bead has been applied and that there are no cracks or areas where caulk is absent. Conduct a similar inspection of the perimeter adhesive every six (6) months thereafter.*
- *Prior to first use, please clean and rinse the bathtub to rid the area of any dust and/or particulates that may remain after installation.*
- *Do not fill the tub with water over the opening of The CleanCut Ultra-Low™.*
- *It is recommended that the unit be cleaned regularly with a gentle household cleaning agent. Do not use harsh cleansers or abrasive tools to clean The CleanCut Ultra-Low™.*
- *It is recommended to completely close the shower curtain prior to turning on water to prevent water from splashing outside the bathtub.*
- *As with any bathtub, children should not be left unattended in the bathtub.*

Safeway Safety Step, LLC, DBA CleanCut
5242 Rialto Road
West Chester, OH 45069